PaxScan® 2520 Receptor, Command Processor and Power Supply

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Manufactured by Varian Medical Systems
 Fabriqué par Varian Medical Systems
 Hergestellt von Varian Medical Systems
 Fabricado por Varian Medical Systems

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Receptor Module

Technical Specifications

Receptor Type	Amorphous Silicon
Conversion Screen	Standard Gd ₂ O ₂ S:Tb Kodak Lanex Screen Optional Cesium Iodide
Pixel Area	Total 19.5 (h) x 24.4 (v) cm (7.68 x 9.6 in) Active 17.9 (h) x 23.8 (v) cm (7.05 x 9.38 in)
Pixel Matrix	Total 1,536 (h) x 1,920 (v) Active 1,408 (h) x 1,888 (v)
Pixel Pitch	127 µm ²
Limiting Resolution	3.94 lp/mm at 7.5 frames per second 1.97 lp/mm at 30 fps
MTF, X-Ray	>35% @ 2 lp/mm with standard screen at 80 kVp
Energy Range	Standard 40 - 150 kVp High kV Option 70 - 1,000 kVp
Fill Factor	57%
Dynamic Range	≥ 2000:1
Spectral Response	400-700 nm; 550 nm peak
Contrast Ratio	Large Area (120 mm): <1% Small Area (10 mm): <7%
Signal Capacity	≥5,000,000 electrons per pixel
Lag	< 12 %, first frame
Gamma	1
Scan Method	Progressive, unidirectional
Data Output	Fiber Optic
A/D Conversion	12-bits ± 1/2 LSB integral linearity
Frame Rate	1-7.5 fps (1x1), 15/30 fps (2x2)
Exposure Window	Internal Sync: 0.9 second External Sync: 200 seconds
Non-Uniformity	1% of full scale maximum after dark current and sensitivity correction inside active area
Temperature Range	10-30°C case temperature. Cooling is required for case temperature >30°C
Power Dissipation	15 watts (50 BTU/hour)
Maximum Entrance Dose	> 4 mR

Mechanical

Size cm (inches)	Standard 26.67 x 31.75 (10.5 x 12.5) High kV Option 65.53 x 42.15 (25.8 x 16.6)
Weight	Standard 4.6 kg (10 lb) High kV Option 26.6 kg (45.5 lb)
Housing Material	Aluminum
Sensor Protection Material	Carbon fiber plate, 2.5 mm thick plus 0.25 mm thick aluminum
Mounting Provisions	Mounting holes 4.19 mm (0.165") diameter

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Command Processor

Mechanical	
Size	6.17 x 27.9 x 25.8 cm (h x w x d) (2.43 x 10.98 x 10.16 inches)
Weight	2.6 kg (6 lb)
Mounting - Bench; mounting holes side faces. 1/4-20 thread, 0.79 cm (0.31") min.	
Control Interfaces	
Ports	1-Ethernet 10BaseT 2-Serial RS-232C 12-opto-coupled inputs, 12-open-collector outputs
Protocols	Ethernet TCP/IP Sockets Serial AIA A15.08/7
Connectors	Ethernet RJ-45 Serial 9-pin D-subminiature (2) I/O 50-pin D-subminiature
Image Acquisition Modes	
Standard Fluoro	768 (h) x 960 (v) (2 x 2 binned) 30 frames per second Progressive Scan
Full-Resolution	1,536 (h) x 1,920 (v) 7.5 frames per second Progressive Scan
Data Matrix Output Fluoro Mode	768 (h) x 960 (v) matrix
Full-Resolution Mode	1,536 (h) x 1,920 (v) matrix
Sync Source	Internal Crystal, External V Drive
Lock Indicator	Front-Panel LED
Main Image Processing	
Non-uniformity Correction	12- bit, gain and offset
Blemish Compensation	Automatic detection Linear nearest neighbor interpolation
Correction Data	Automatic acquisition, One set per scan rate
Digital Video Output - 12-bit parallel image data, Pixel clock, Data valid, H Sync, V Sync	
Connector	50-pin SCSI-2
Optional Automatic Brightness Stabilization (ABS)	
ABS output	0-20 volt 15 mA, complimentary pair
Connector	Female, 15-pin D-subminiature
X-Ray Generator	± 10 volt, ± 15 mA
Analog Video Output Scan Conversion	Can be factory set to one of: 525 line / 60 Hz / interlaced 625 line / 50 Hz / interlaced 1,049 line / 60 Hz / interlaced 1,249 line / 50 Hz / interlaced
Recursive Filtering	Single-pole, Range 0 to 100%
Signal Detection	Digital windowed histogrammer
Analog Video Output Connector	BNC and 15-pin D-subminiature
Gamma Correction	12- to 8-bit programmable lookup table
Digital Video Output	8-bit parallel
Last Image Hold	Automatic on contact closure

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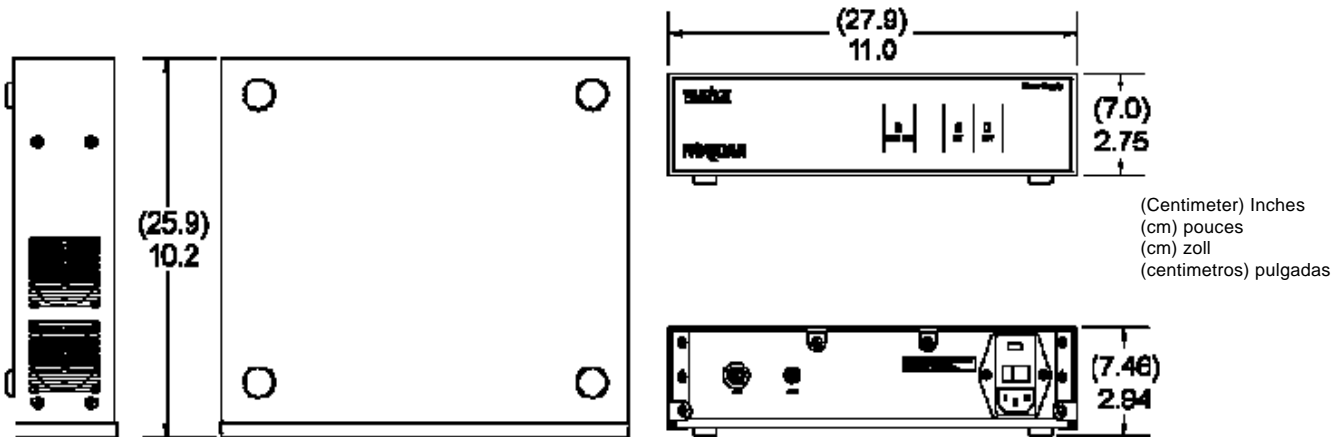
This document contains a technical description of an x-ray imager. The structure and capabilities of the equipment described herein are subject to change without notice. No representations are made by this document concerning the feasibility or status of clearance for medical use of this equipment in the United States or any other jurisdiction.

<p>VARIAN medical systems</p>	<p>X-RAY PRODUCTS</p>
<p>Salt Lake City, UT Charleston, SC</p>	<p>1-801-972-5000 1-843-767-3005</p>
<p>www.varian.com</p>	